**CZ**

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Zdravotnícke pracovisko: HILDING ANDERS CUREM I., II.:

INSTRUCTIONS FOR USE AND MAINTENANCE

FOR HILDING ANDERS TROPICO, SPIRIT AND CUREM PASSIVE ANTI-DECUBITUS MATTRESSES

Thank you for purchasing one of our mattresses. Hilding Anders TROPICO, SPIRIT and CUREM mattresses are designed as Class I medical devices and are thus intended for use in hospitals and institutions, as well as at home. They are intended for long-term interior-only use.

Intended purpose and materials

name	Models	materials	intended purpose ¹
HILDING ANDERS TROPICO I.	Topper Visco, Topper Flexi, Topper TropicoGuard Medical AERO, Topper Visco Medidry, Topper Aegis, Latex Lila, Sleepguard Latex, Riviera Plus, Maxi, Tropico Plus F85 Maxi, Sol, Nika, Amundsen, Arabela, Arabela Hard, Austin Air Geltouch, Austin Air Latex, Austin Air Visco, Alpine Blue Air, Medixo Airspring Bluegel, Maxi Kings, Enigma, Tropico Guard Ortho, Heureka Plus, Heureka Plus Flexi, Heureka Plus Visco, Terno Flexi, Terno Visco, Triker Flexi, Triker Visco, Romantika Kašmír, Kolos, Kolos Bio Ecology, Kontakt Bambus, Dáša Tropico, Dáša Tropico Visco, Latex Supreme, Atlas, Atlas Visco, Atlas Flexi, Atlas, Astana 3D Flex, Atlas Astana 3D Visco, Atlas Spring, Atlas Pocket, Atlas Pocket Spring, Super Fox Blue, Super Fox Cloud, Super Fox Visco, Super Fox Visco Heaven, Swisslab Active, Swisslab Natur, Swisslab Big Boy, Swisslab Prestige XD, Swisslab Prestige Gel, Petra Tropico, Klára Tropico, Šárka Tropico, Dana, Vojta, Vojta Travel	foam: HR, Flexi, Visco, Hybrid, Gel, Coconut coir, Latex, RE foam. spring: Bonell, Pockets, Multipockets	Passive anti-decubitus mattress designed for patients with a low risk of pressure ulcers (Norton scale 19-16).
HILDING ANDERS TROPICO II.	Tropico Guard Air Hybrid, Tropico Guard Antibacterial, Tropico Guard Antibacterial Heaven, Tropico Guard Medical, Tropico Guard Medical Heaven, Stream, Stream Kokos, Forest Air	foam: HR, Flexi, Visco, Hybrid, Gel, Coconut coir, Latex, RE foam. spring: Bonell, Pockets, Multipockets	Passive anti-decubitus mattress designed for patients with a medium risk of pressure ulcers (Norton scale 16-14).
HILDING ANDERS SPIRIT I.	Topper Spirit Superior Root, Spirit Superior Cloud, Spirit Superior Comodore, Spirit Superior Triumph, Spirit Superior Nucleus	foam: HR, Flexi, Visco, Hybrid, Geltouch, Latex, X – Dura, Geltouch spring: Bonell, Pockets, Multipockets	Passive anti-decubitus mattress designed for patients with a low risk of pressure ulcers (Norton scale 19-16).
HILDING ANDERS SPIRIT II.	Spirit Superior Latex, Spirit Superior Visco, Spirit Excelsior Oasis, Spirit Jaguar	foam: HR, Flexi, Visco, Hybrid, Geltouch, X-Dura. spring: Bonell, Pockets, Multipockets	Passive anti-decubitus mattress designed for patients with a medium risk of pressure ulcers (Norton scale 16-14).
HILDING ANDERS CUREM I.	Curem C2000 II Style, Curem C3000 HY, Curem C3500, Curem C4500, Curem Dream, Curem Dream Forte, Curem H6 Robust, Curem H6 Velvet, Curem H6 Festo, Curem H6 Passion, Curem H6 TRANQUILIT, Curem H6 BRACE, Curem Ultima Heaven, Curem H6 Voyager, Curem H6 Ego	foam: HR, Flexi, Visco, Hybrid, Geltouch, X-Dura	Passive anti-decubitus mattress designed for patients with a low risk of pressure ulcers (Norton scale 19-16).
HILDING ANDERS CUREM II.	Curem H6 Gelfeel, Curem C7000 XD	foam: HR, Flexi, Visco, Hybrid, Geltouch, X-Dura	Passive anti-decubitus mattress designed for patients with a medium risk of pressure ulcers (Norton scale 16-14).

¹ Decubitus = bedsore

Information about the specific size and load bearing capacity can be found on the packaging label of each mattress. These specifications vary depending on the model.

The manufacturer retains the right to colour deviations in the materials used that do not affect the performance of the product. Differences in the firmness of the foams are also permissible as long as they remain within the prescribed ranges. Mattresses with reinforced edges and some mattresses larger than 90 x 200 cm are delivered with glued structural joints that do not affect the performance of the product. Due to the standard width of fabrics, mattresses larger than 90x200 cm may be delivered with a different cover stitching pattern that does not affect the performance of the product. A block of quality hybrid foam is glued to all types of mattresses (including latex) when lengthened, meaning longer than 200 cm.

For hygiene reasons, a purchased mattress cannot be exchanged or returned once used. The manufacturer is not responsible for any problems caused by choosing an unsuitable mattress firmness or not respecting the manufacturer's instructions. Before buying a mattress, consult with the attending physician, if this is desirable in view of the health condition of the user.

Indications:

Prevention of pressure ulcers – degree of risk according to the intended purpose stated above.

Contraindications:

Unstable fractures, burns, extensive swelling

Before first use:

Our mattresses may give off a characteristic aroma after being unpacked, however this is completely safe and will disappear after a few days. Allow compressed mattresses to rest without loading for at least 72 hours after you remove them from the packaging. Note that mattresses that include memory (visco) foam (meaning mattress models that have "visco" (memory foam) in their name or in the materials specification) may exhibit slight deformation/shrinkage due to temperature changes during transport. After unpacking your mattress, allow it to "breathe" at room temperature for around 24 hours – we also recommend giving it a gentle shake. The mattress will return to its original size after it reaches room temperature.

Mattress cover:

Our mattresses are equipped as standard with a knitted or woven breathable textile cover (its washability, composition and construction differ depending on the specific mattress type). They may also be equipped with a coated polyurethane (PU) cover. This type of cover is impermeable to liquids yet remains permeable to vapour, enabling both more effective home care and also making this variant suitable for professional use in healthcare, nursing etc. The PU cover is non-slip, suitable for allergy sufferers and asthmatics, can be wiped down and is washable at high temperatures (up to 95°C). It features a zip for easy removal and installation, protected by a flap to keep it clean and prevent the ingress of dirt to the mattress core.

Mattress use:

Our mattresses may be used for home care on both a standard bed with a slatted frame, and an adjustable bed. The positioning options of the mattress may be reduced if it has reinforced sides. A slatted base will enhance the orthopaedic properties of the mattress. Mattresses for hospital care are designed to be used with standard adjustable beds.

Before unpacking the mattress, check that the size corresponds to the internal dimensions of the bed. Place mattresses with a memory side so that this side is always uppermost. With double-sided mattresses (where the sides have differing firmness and/or construction), either side may be used depending on the need for firmness. If a mattress does not have a firmer/softer side, always place it with the profiled (trimmed) side uppermost! Use mattresses only with their originally supplied covers, otherwise optimal functionality cannot be guaranteed. Replacement covers can be purchased separately.

Mattress placement (what goes under the mattress):

To ensure your mattress provides ideal orthopaedic properties and optimal airiness, it is important to select a suitable base that corresponds to the mattress construction. This will also ensure the longest possible lifespan for your mattress. By selecting a quality base (slatted frame, bed) you will ensure proper ventilation and the longest possible mattress lifespan. Sprung mattresses must be placed on a solid base (with sufficient ventilation) or a slatted (firm) base, with the gaps between the slats being no wider than 4 cm. Sandwich and latex mattresses are suitable for use on a slatted base, which will enhance the orthopaedic properties of the mattress, but they can also be placed on a slatted frame (firm). A slatted (firm) base must always have at least 24 sprung slats, and the gaps between them must not exceed 4 cm! Foam and latex mattresses (except ones reinforced with coconut/latex board) are suitable for adjustable frames, whether mechanical or motorised. Do not place mattresses on wire or metal frames unless stated otherwise on the packaging leaflet or mattress label.

Mattress care:

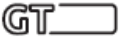
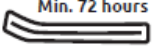
The mattress core can be cleaned as needed using dry foam or waxing (wet foam with reduced moisture followed by drying). You can also lightly and evenly apply bicarbonate of soda, leave it to act for several hours, and then thoroughly vacuum the mattress. Keep in mind that mattresses absorb body moisture. It is important that the mattress is thoroughly ventilated and, if possible, kept uncovered.

When maintaining all types of covers, always follow the care symbols sewn into the hem (at the zip). Do not use a fabric softener. Never iron covers, and do not use caustic, corrosive or abrasive substances or objects to remove stains and dirt! Clean or wash the mattress cover immediately after contamination or if it gets dirty. You can also use a spare cover, which can be purchased separately. Make sure that no foreign object gets between the mattress, respectively the mattress core, and the cover, or between the mattress and the patient's skin! We recommend using a mattress protector to extend the product's lifespan. Turn the mattress over twice a year.

Product symbols:

	Read the instructions for use		A mark indicating compliance with European regulations
	Unique device identifier		Medical device
	The product can be tumble dried at a lower temperature (60°C)		The product must not be dried in a tumble dryer
	Max. washing temperature 30°C (textile bag)		Max. washing temperature 60°C (textile bag)
	Max. washing temperature 40°C (textile bag)		Max. washing temperature 95°C (textile and PU – polyurethane bag)
	Iron at max. temperature 110°C (textile bag)		Do not iron (PU polyurethane bag)
	Professional dry cleaning		The product must not be dry cleaned
	The product must not be bleached		Mattress load capacity
	Protect from moisture		Protect from sunlight
	Double-sided mattress		Symmetrical mattress construction (1=2) Single-sided use

Other symbols:

	Flexifoam foam		HR cold foam
	Geltech foam		Visco foam
	Hybrid foam		Bonell springs
	Pocket springs		Multipocket springs
	Coconut coir		X-Dura foam
	Latex		Two-piece cover
	Reinforced side		Non-glued core
	Mattress size		RE (recycled) foam
SOFT	Softer side of the mattress	HARD	Firmer side of the mattress
	Min. 72 hours without loading after removal from packaging		Storage in a rolled (compressed) state for max. 30 days

Storage, basic conditions of use:

Compressed mattresses can be stored in a compressed state for a maximum of 30 days. Be as careful as possible when removing the packaging to ensure you do not cut too deeply and damage the mattress! Uncompressed mattresses can be used immediately after unpacking. Never fold mattresses. Mattresses with pocket springs must be transported and stored horizontally. Store in a dry place at a temperature of over 5°C and protect from freezing. Do not expose mattresses to direct sunlight and temperatures of over 50°C, even if in their original packaging. Avoid sudden differences in temperature and humidity – If you need to place the product in a very dry environment, you should let it acclimatise – here, too, avoid sudden changes in humidity. Store mattresses and other products of a similar nature away from sources of ignition, and do not dispose of them in a fire! Use a powder or foam fire extinguisher to put out small, local areas of burning. The materials used in our mattresses comply with standards as recorded in the technical data sheet of the material used. Keep the packaging out of the reach of children! The mattress upholstery layer – single- or multi-layer – is made of LDPE (Low Density PolyEthylene) and is recyclable.

Disposal of the product and its packaging; environmental protection:

Dispose of the mattress and its components in accordance with local laws and regulations. Mattresses should be disposed of in landfills – their individual components, including the packaging, are recyclable. Dispose of in accordance with the Act on Waste.

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IČO/ID: 2617 32 12, DIČ/VAT: CZ26173212, bank: 212 591 826/0300, ID DS: bxgeekq

e:mail: info.cz@hildinganders.com; tel.: +420 481 587 410; web: www.hilding.cz

If a defect appears in a purchased product during the warranty period, the customer has the right to make a claim concerning such defect. A defect means a change (in the properties) in a product due to a production defect or a defect in the material used to produce the product. The conditions are detailed on the manufacturer's website.

The following cannot be considered a defect: a change (in the properties) in a product that arises during the warranty period as a result of normal wear and tear, rippling of the cover or a change in the size of the cover caused by normal use, or increased sweating on the mattress (especially for mattresses with memory – lazy – foam, which by their nature feature higher heat accumulation and can thus cause an increased feeling of warmth and thus sweating).

The following also cannot be considered a defect: a deviation in size within the acceptable tolerance of +/- 10 mm. Mattress size tolerance when placed on a base/bed (e.g. a bed with non-sunken mattresses): width and length +/- 20 mm, height +/- 10 mm. Mattress core/foam firmness tolerance: +/- 15% (compression resistance in kPa). For mattresses with pressure-bonded (recycled) "RE" foam, a firmness deviation of +/- 20% is permitted in view of the manufacturing technology for that foam. Sagging (deformation and changes in properties) within acceptable tolerances for mattresses made of polyurethane (Flexifoam®, PUR, cold, visco, hybrid, etc.) foams, in particular sagging that does not exceed 15% of the total height of the mattress. Colour deviations in the foam and upholstery fabrics compared to the material sample and cover material designs or stitching patterns that do not impact the performance of the product. For other than PU (impermeable) covers (in particular those with a high polyester fibre content), we do not recommend using terry and cotton bed linen, as this can result in pilling. We always recommend the use of a mattress protector in such cases.

Warranty certificate, general warranty conditions: The Hilding Anders Česká republika a.s. warranty for product defects is provided to the buyer in accordance with generally applicable legislation for a period of 24 months or, in the cases listed below, for an extended period. The basic prerequisite for a proper claim is demonstration that the product was purchased from a retailer, ideally through the presentation of proof of purchase (receipt, invoice, etc.) and the presentation of the claimed product with a proper description of the defect, or the warranty certificate (especially for warranties of over 24 months). For this reason, please keep in a safe place the warranty certificate, proof of purchase and product label included in the packaging of each product. The buyer must properly inspect the product upon receipt. If they find a defect in the product, they must notify the retailer of this fact without undue delay. The retailer bears no responsibility for defects caused during the transport of a product using the buyer's means of transport, or for any later claim regarding the quantity delivered or any damage to the products not indicated in the document demonstrating receipt of the products. Contact details for making a claim are listed on the sales receipt. In the event of a claim, please proceed in accordance with the retailer's Complaints Policy, which can be found on the retailer's website or directly at the retailer.

Statutory warranty: Provided that the product has been properly used and cared for in accordance with these Instructions for Use, a right from a defect (claim) must be exercised by the customer no later than 24 months (unless stated otherwise) from receipt of the product while, unless the product was personally collected at the premises of the retailer, receipt of the product means the moment when the customer accepted the product from a carrier. The duration of a right from a defective performance ("Warranty Period") will be extended by the time during which the product undergoes a warranty repair until such time when the buyer, after the completion of the complaints procedure, must collect the product. The Warranty Period should not be confused with the lifespan of the product, meaning the time the product should last with proper use and proper care, including maintenance, taking into account its properties, the given purpose and differences in intensity of use. The Warranty Period is a general administrative period without any direct connection with the usual lifespan of a specific product. Upon expiration of the Warranty Period, the buyer's right to make a warranty claim also expires. The warranty does not cover normal wear and tear to the mattress caused through normal use. Furthermore, the liability for defects in products does not apply to defects that the buyer was notified of by the retailer at the time of purchase (conclusion of the contract) if the buyer did not refuse to accept the product at delivery. A right from a defective performance also cannot be exercised for a product sold at a reduced price when such reduced price was arranged precisely due to the presence of a defect that was brought to the attention of the buyer before the purchase.

Extended warranty: The manufacturer provides buyers with an extended warranty for mattress core defects under the following conditions, while the duration of the warranty is indicated on the label or sticker on the mattress packaging. For selected mattresses, an extended warranty is provided for mattress core defects, subject to the placement of the mattresses on a flexible base (slatted frame), except for sprung mattresses. This extended warranty benefit is free of charge – you only need to register your mattress. It is important to register no later than 30 calendar days from the date of purchase of the mattress at <https://reg.hilding.cz>. To register, you only need to enter the mattress name, size and serial number (SN code) - you can find this on a paper self-adhesive label located on one of the shorter sides of the mattress. More details about the extended warranty can be obtained from the retailer. The extended warranty only applies to the mattress core. If this is replaced, the new one is guaranteed for the duration of the remaining extended warranty, but no less than 24 months.

Extended regressive warranty for Curem and Spirit mattresses: The manufacturer provides customers with an extended "Regressive" warranty for mattress core defects for selected Curem and Spirit mattresses, which can be up to 10 years and is indicated on the label on the mattress packaging. The extended warranty applies only to the mattress core. From 0 to 6 years, the customer has the right to full payment (repair or replacement), and this is reduced by 20% every year over the next 6 years. This extended warranty is subject to registration of the buyer and mattress at <https://reg.hilding.cz>.

By purchasing a product (mattress), the buyer confirms they have been duly acquainted with the retailer's Complaints Policy. If you wish to make a claim regarding a mattress, it must be complete and whole, must be in a hygienically acceptable condition (meaning clean, dry, hygienically safe, etc.), if possible in the original packaging or replacement packaging suitable for transporting the product (note: a thin layer

of stretch wrap is not considered suitable packaging for transport!). The product must not be folded or rolled up, with the exception of compressed mattresses. If a returned mattress is unhygienic (dirty, wet, smelly, etc.), the complaints procedure will be interrupted until the buyer rectifies such problem with the mattress, while such fact will be entered into the claim record. If, however, the buyer does not present a hygienically safe mattress within 1 month from the date of interruption of the complaints procedure, the claim will be considered terminated as unreasonable.

Report any serious incident that has occurred in connection with the device in question to the manufacturer and the competent authority.

Last revision of this document: 2025-05-22, rev. 10, language version EN

Hilding Anders Česká republika a.s. has prepared questionnaires for the purpose of collecting clinical data within the framework of PMCF (Post-Market Clinical Follow-up). The aim of data collection is to confirm the safety and effectiveness of anti-decubitus mattresses and monitor potential risks and adverse effects. This assessment is an important criterion that must be met in accordance with the legislative requirements for medical devices. The data obtained from the questionnaires will be used to update the clinical evaluation in accordance with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

Home use: HILDING ANDERS TROPICO I., II.

Healthcare facility: HILDING ANDERS TROPICO I., II.

Home use: HILDING ANDERS SPIRIT I., II.

Healthcare facility: HILDING ANDERS SPIRIT I., II.

Home use: HILDING ANDERS CUREM I., II.

Healthcare facility: HILDING ANDERS CUREM I., II.

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