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Zdravotnícke pracovisko:

INSTRUCTIONS FOR USE AND MAINTENANCE

OF ANTIDECUBIT MATTRESSES OF HILDING ANDERS MEDICAL SERIES

Thank you for buying a mattress from our manufacture. Mattresses of Hilding Anders Medical series are medical devices of risk class I, both for hospital or institutional use and home use. They are solely intended for long-term use in indoor conditions.

Intended purpose and material used

Designation	Model	Material Used	Intended purpose ¹
HILDING ANDERS MEDICAL I	Isabela, Elastix	foam: HR, Flexi, Visco	Passive antidecubit mattress designed to support the prevention of pressure ulcers at low risk of developing, the Norton Scale of 19-16.
HILDING ANDERS MEDICAL II	Hospital T, Hospital R, Delux Mono, Delux Duo	foam: HR, Flexi, Visco, Hybrid	Passive antidecubit mattress designed to support the prevention of pressure ulcers at medium risk of developing, the Norton Scale of 16-14.
HILDING ANDERS MEDICAL III	Delux Visco, Supra Care Visco, Supra Care Geltouch, Air Clinic Visco, Air Extrem Visco, Clinix, Ebers	foam: HR, Flexi, Visco, Hybrid, Geltouch	Passive antidecubit mattress designed to support the prevention of pressure ulcers at high risk of developing, the Norton Scale of 14-12.
HILDING ANDERS MEDICAL IV	Ergonomix Visco, Supra Module, Supra Individual, Module Zone	foam: HR, Flexi, Visco, Hybrid, Geltouch, X-Dura	Passive antidecubit mattress designed to support the prevention of pressure ulcers at very high risk of developing, the Norton Scale of <12.

¹decubitus = pressure

Please refer to the package label of each mattress for the information on the specific dimensions and max. weight capacity (load capacity) of the mattress. These parameters vary depending on the individual customer's requirements.

The manufacturer reserves the right to colour variations of the materials used that do not affect the performance characteristics of the product. Differences in the stiffness of the foams within the prescribed standards are also permissible. Mattresses with reinforced edges and some mattresses larger than 90 x 200 cm are delivered with a glued structural joint, which does not affect the performance characteristics of the product. The manufacturer is not responsible for any problems caused by choosing an inappropriate mattress stiffness or failing to follow the manufacturer's instructions.

For hygienic reasons, the purchased mattress cannot be exchanged or returned once used. Due to the product's nature, the mattress shall be chosen by the professional medical staff. Please consult your doctor before buying a mattress, even if it is to be used at home.

Contraindications:

Unstable fractures, burns, and extensive swelling

Before first use:

Upon unpacking, the mattress can have a characteristic smell, which is not harmful to health and disappears within a few days. After unpacking, leave a compressed mattress without load for at least 72 hours. Uncompressed mattresses can be used immediately after unpacking. Mattresses with memory foam (Elastix, Delux Visco, Air Extrem Visco, Air Clinic Visco, Ergonix Visco, and Supra Module) may experience slight deformation/shrinkage due to different temperatures during transport. After unpacking, leave the mattress without load at room temperature for approx. 24 hours; we also recommend plumping it up a bit. The mattress regains its original size after warming up to room temperature.

Mattress cover:

The mattresses come with a polyurethane-laminated fabric cover as a standard. The cover is waterproof, vapour permeable, non-slip, washable and suitable for machine washing at high temperatures (up to 95 °C). It is suitable for allergy sufferers and asthmatics. It has a double-sided lapped zipper to prevent contamination of the zipper and the penetration of dirt into the mattress core. Alternatively, it can also be made of knitted textile materials, washable at a higher temperature (up to 60 °C).

Use of the mattress:

The mattress is solely intended for use with bed slats or in regular positioning beds (if a mattress with reinforced sidewalls is used, the possibility of positioning may be reduced). A slatted grid supports the orthopaedic properties of the mattress. Before unpacking the mattress, make sure that the dimensions of the mattress correspond to the internal dimensions of the bed. Always place the mattress with its memory side up. With double-sided mattresses (with different stiffness and/or side construction), both sides can be used depending on the stiffness required. If the mattress does not have a harder/softer side, always place it with the profiled (cut) side up! Only use the mattresses with the initially supplied cover; otherwise, optimal functionality cannot be guaranteed. A replacement cover can be purchased separately. Ensure no foreign object gets between the mattress, the cover and the patient's skin!

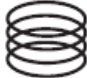
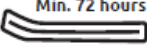
Recommended care:

The mattress core can be cleaned with dry foam or by waxing (wet foam with reduced wetting and drying) as needed; alternatively, dust it lightly and evenly with baking soda, let it act for several hours and vacuum thoroughly. The polyurethane cover can be washed at a maximum temperature of 95 °C; the textile cover can be washed at a maximum temperature of 60 °C. **Do not use fabric softener.** Do not iron the cover. Do not use caustic, corrosive or abrasive substances and objects for cleaning! Always follow the care symbols sewn into the hem of the cover (to the zipper), which are explained below:

Product symbols:

	Read the instructions for use		A mark indicating compliance with European regulations
	Unique device identifier		Medical device
	The product can be tumble dried at a lower temperature (60°C)		The product must not be dried in a tumble dryer
	Max. washing temperature 30°C (textile bag)		Max. washing temperature 60°C (textile bag)
	Max. washing temperature 40°C (textile bag)		Max. washing temperature 95°C (textile and PU – polyurethane bag)
	Iron at max. temperature 110°C (textile bag)		Do not iron (PU polyurethane bag)
	Professional dry cleaning		The product must not be dry cleaned
	The product must not be bleached		Mattress load capacity
	Protect from moisture		Protect from sunlight
	Double-sided mattress		Symmetrical mattress construction (1=2) Single-sided use

Other symbols:

	Flexifoam foam		HR cold foam
	Geltech foam		Visco foam
	Hybrid foam		Bonell springs
	Pocket springs		Multipocket springs
	Coconut coir		X-Dura foam
	Latex		Two-piece cover
	Reinforced side		Non-glued core
	Mattress size		RE (recycled) foam
SOFT	Softer side of the mattress	HARD	Firmer side of the mattress
	Min. 72 hours without loading after removal from packaging		Storage in a rolled (compressed) state for max. 30 days

Storage, basic conditions for use:

Do not bend the mattress. Keep it in a dry place and store it at a temperature of 5-50 °C. Do not expose to direct sunlight. Keep completely away from sources of ignition. Do not throw it into fire! Use powder or snow fire extinguishers to extinguish a minor local fire.

Product and packaging disposal; environmental protection:

Dispose of the mattress by depositing it at a dumping ground; the individual components of the mattresses are recyclable. The packaging is recyclable.

GENERAL WARRANTY CONDITIONS

If a defect occurs in the purchased goods during the warranty period, the customer has the right to claim this defect. A defect means a change (of a property) of the goods caused by a manufacturing defect or a defect in the material used to manufacture the goods. The conditions are detailed on the manufacturer's website.

Among other things, a change (of a property) of the goods that occurred during the warranty period as a result of its normal wear and tear, curling of the cover or changes in the size of the cover caused by normal use of the mattress, as well as increased sweating on the mattress (especially on mattresses with memory - lazy - foam, which accumulates heat more by its very nature and can thus cause an increased feeling of heat and sweating) **cannot be considered a defect.**

Furthermore, the following cannot be considered a defect: dimensions within the permissible tolerance of +/- 10 mm; Tolerance of the size of the mattress placed on the surface of the base/bed (e.g. beds with non-recessed mattresses): width and length +/- 20 mm; height +/- 10 mm. The permissible tolerance of the mattress core/foam stiffness is +/- 15% (compression resistance in kPa). A deviation in stiffness of +/- 20% is allowed for mattresses with pressure-bonded (recycled) "RE" foam due to the production technology of the given foam. Sagging (or deformation and changes in properties) that are within the permitted tolerances for mattresses made of polyurethane foams (Flexifoam®, PUR, cold, Visco, hybrid, etc.), especially sagging that does not exceed 15% of the total height of the mattress. Deviations in the colour tone of foams or upholstery fabrics from the fabric swatches or upholstery fabric and stitching patterns that do not affect the performance characteristics. We do not recommend the use of terrycloth and cotton bed linen for covers other than PU - impermeable (especially those with a high content of polyester fibres) as it might cause pilling of the cover fabric. With these product types, we always recommend providing the mattress with a mattress protector.

Guarantee statement, general warranty conditions Warranty for product defects is granted by Hilding Anders Česká republika a.s. to buyers in accordance with generally applicable legal regulations for a period of 24 months or for an extended period in the following cases. The basic prerequisite for a due complaint is proving the purchase of the products from the seller, preferably by presenting a proof of purchase (such as a receipt or an invoice) and submitting the claimed product with a description of the defect or a warranty card (especially in the case of a warranty longer than 24 months). Therefore, please carefully keep the warranty card, proof of purchase of the product and the product label, which is a part of the packaging of each product, in case of a possible complaint. The buyer is obliged to check and inspect the goods upon receipt properly. Should they find any defect in the product, they are obliged to notify the seller of this fact without undue delay. The manufacturer is not responsible for defects caused by the transport of the goods by the buyer's means of transport, as well as for later complaints about the quantity delivered or damage to the shipment that was not indicated on the receipt of the goods. The contact details for making a claim are listed on the sales receipt. In the event of a complaint, please proceed in accordance with the seller's Complaints Terms and Conditions, which you can find on the seller's website or obtain directly from the seller.

Statutory guarantee: Assuming that the principles of correct product use and care are followed in accordance with these Instructions for Use and Maintenance,

the buyer shall exercise their right to claim a defect (complaint) within 24 months (unless otherwise stated) from the product takeover when buying from the seller; if the product was not personally collected from the seller, taking the product from the carrier is considered the moment of takeover. The duration of the right from defective performance (hereinafter also the warranty period or warranty) is extended by the time during which the product was under warranty repair until the time when the buyer was obliged to take over the product after the end of the complaint procedure. The warranty period cannot be confused with the lifetime of the product, i.e. the time for which, with proper use and care, including maintenance, the product can last based on its properties, intended use and differences in the intensity of its use. The warranty period is a general administrative period that has no direct connection with the usual lifetime of a specific product. The buyer's right to claim the warranty expires upon expiration of the warranty period. The warranty does not cover normal wear and tear of the mattress caused by its usual use. Furthermore, liability for product defects does not cover defects of which the seller notified the buyer at the time of purchase (contract), whereas the buyer did not refuse to accept the product upon delivery. Furthermore, the right from defective performance cannot be applied to a product sold at a reduced price, for which the reduced price was negotiated with respect to the said defect that the buyer was made aware of before purchasing the product.

Report any serious incident that has occurred in connection with the device in question to the manufacturer and the competent authority.

The latest revision of this document: 2025-04-15, rev. 4, EN language version

Hilding Anders Česká republika a.s. has prepared questionnaires for the purpose of collecting clinical data within the framework of PMCF (Post-Market Clinical Follow-up). The aim of data collection is to confirm the safety and effectiveness of anti-decubitus mattresses and monitor potential risks and adverse effects. This assessment is an important criterion that must be met in accordance with the legislative requirements for medical devices. The data obtained from the questionnaires will be used to update the clinical evaluation in accordance with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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