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Lekárske pracovisko:

INSTRUCTIONS FOR USE AND MAINTENANCE

HILDING ANDERS AID POSITIONING AIDS

Thank you for purchasing a positioning aid of our production. The Hilding Anders AID series of positioning aids are medical devices of risk class I, designed for both hospital and institutional use, as well as for household use. They are designed for long-term use exclusively indoors.

Intended purpose and material used

Name	Models	Material used	Intended purpose
HILDING ANDERS AID	Bederník, Podkoleník, Podhlavník, Tom, Tom Aloe, Tom Kokos, Tom Gel, TG Antibacterial Gel, Talisman Geltouch, Rehabilitační válec, Medical, Aegis, Micro, Medical Micro, Rizo PU, Tropic Hypoallergen, Tropic Hypoallergen Molton, Tropic Polycotton Medical, Tropic Polycotton Medical Molton, Tropic PU Protect, Tropic PU Protect Molton, Spa Medical	Foam: HR, Flexi, Visco, Hybrid, Gel, Coconut fiber, Latex, RE foam.	HILDING ANDERS AID positioning aids are designed to support the prevention of pressure ulcers in care and healthcare facilities in patients with limited mobility.

¹ Pressure ulcer = bedsore

Information about the specific size of the positioning aid can be found on the label of the packaging of each aid.

The manufacturer reserves the right to color deviations of the materials used that do not affect the utility properties of the product. Differences in the stiffness of foams within the prescribed standards are also permissible.

For hygienic reasons, the purchased positioning aid cannot be replaced or returned after use. Due to the nature of the product, the choice of the device is determined by professional medical staff. The manufacturer is not responsible for any difficulties caused by the selection of an unsuitable device or by not following the manufacturer's instructions. Before purchasing the device, consult the attending physician if this is desirable with regard to the user's health condition.

Indications:

The medical device is suitable for supporting correct positioning and ensuring comfort in immobile patients, thus helping to reduce the risk of pressure ulcers during prolonged lying down.

Contraindications:

There are no known contraindications. It must be used in such a way as to prevent hyperextension and contractures.

Before first use:

Positioning aids of our production can emit a characteristic aroma after unpacking, which, however, is completely harmless to health and disappears after a few days.

Positioning aid cover:

Positioning aids are equipped with a knitted or woven breathable textile cover as standard; they can also be provided with a polyurethane (PU) coating with a coating; where such a cover is impermeable to liquids and at the same time permeable to couples and not only allows for more effective home care, but also ensures the suitability of the variant for professional use in health care and nursing, etc. The PU cover is non-slip, suitable for allergy sufferers and asthmatics, washable and washable at high temperature (up to 95°C); It has a zipper for easy removal and threading, protected by a flap (drip tray) against contamination of the zipper and the ingress of dirt into the core of the positioning aid.

How to use:

The positioning aid can be used for home care in both a regular bed with a bed with a bed and an adjustable bed.

Only use the positioning aid with the cover originally supplied; Otherwise, optimal functionality cannot be guaranteed. A replacement cover can be purchased separately.

Positioning aid:

The core of the positioning aid can be cleaned as needed with dry foam, or vaxation (wet foam with reduced humidity with drying), or lightly and evenly dusted with baking soda, left on for several hours and vacuumed thoroughly. Keep in mind that the device absorbs body moisture. It needs to be thoroughly ventilated, if possible, do not cover the aid.

When maintaining all types of covers, always follow the care symbols sewn into the hem (to the zipper). Do not use fabric softener. Never iron the covers and do not use corrosive, corrosive or abrasive substances and objects to remove stains and dirt! Clean or wash the cover immediately after it becomes contaminated or otherwise contaminated. You can also exchange this for a replacement cover, which can be purchased separately. Make sure that no foreign object gets between the core of the device and the cover, nor the user's skin!

Symbols on the product:

	Read the instructions for use		A mark indicating compliance with European regulations
	Unique device identifier		Medical device
	The product can be tumble dried at a lower temperature (60°C)		The product must not be dried in a tumble dryer
	Max. washing temperature 30°C (textile bag)		Max. washing temperature 60°C (textile bag)
	Max. washing temperature 40°C (textile bag)		Max. washing temperature 95°C (textile and PU – polyurethane bag)
	Iron at max. temperature 110°C (textile bag)		Do not iron (PU polyurethane bag)
	Professional dry cleaning		The product must not be dry cleaned
	The product must not be bleached		Mattress load capacity
	Protect from moisture		Protect from sunlight
	Double-sided mattress		Symmetrical mattress construction (1=2) Single-sided use

Other symbols:

	Flexifoam Foam		HR Cold Foam
	Geltech foam		Visco foam
	Hybrid foam		X-Dura foam
	Coconut fiber		Latex

Storage, basic conditions for use:

When removing the packaging, proceed with the utmost care so as not to cut the packaging too much and not damage the cover of the aid! Store in a dry place, protect from frost and temperatures up to 5 °C. Do not expose the positioning aid, even if originally packaged, to direct sunlight and temperatures above 50 °C, and also avoid sharp fluctuations in temperature and humidity – if you place the product in too dry areas, it is advisable to at least "acclimatize" it – so again: avoid a sudden change in humidity. Keep the positioning aid as well as other products of a similar nature completely away from sources of ignition, do not throw them into fire! In the case of liquidation of minor local ignition, use powder or snow fire extinguishers. The materials used meet the standard requirements of the standards according to the technical data sheet of the material used. Keep the packaging out of the reach of children! The packaging – single or multi-layer – is made of LDPE (Low Density PolyEthylene) film and is recyclable.

Disposal of the product and its packaging; Nature protection:

Dispose of the positioning aid and its components in accordance with local laws and regulations. Positioning aids are disposed of by landfilling, its individual components, including the packaging, are recyclable. Disposal should be carried out in accordance with the Waste Act.

GENERAL WARRANTY CONDITIONS

If a defect occurs in the purchased goods during the warranty period, the customer has the right to complain about this defect. A defect is a change (property) of goods caused by manufacturing defects or defects in the material used to produce the goods. The conditions are detailed on the manufacturer's website.

A defect cannot be considered, inter alia, a change (properties) of the goods that occurred during the warranty period as a result of their normal wear and tear, wavy of the cover or a change in the size of the cover caused by normal use of the device, as well as increased sweating on the device (especially in the case of aids with memory – lazy – foam, which by its nature excels in higher heat accumulation and can thus cause an increased feeling of heat and thus sweat).

The following cannot be considered as a defect: dimensions within the permissible tolerance +/- 10 mm; Tolerance of the dimension of the positioning aid: width and length +/- 20 mm; height +/- 10 mm. The permissible tolerance of core/foam stiffness is +/- 18 % (compression resistance in kPa). Colour deviations of foams and upholstery fabrics in colour tone against the fabric pattern and the patterns of upholstery fabrics or stitching patterns of upholstery fabrics that do not affect the utility properties. For covers other than PU - impermeable (especially those with a high content of polyester fibers), we do not recommend the use of terry and cotton bedding, the cover may pill.

Warranty card, general warranty conditions: The warranty for defects in the products of Hilding Anders Česká republika a.s. is provided to the buyer in accordance with generally applicable legal regulations for a period of 24 months, or in the cases listed below, in its extended length. The basic prerequisite for a proper complaint is to prove the purchase of the products from the seller, preferably by submitting proof of purchase of goods (receipt, invoice, etc.) and submitting the claimed product with a description of the defect or a warranty card (especially in the case of a warranty longer than 24 months). For this reason, for any complaint, please carefully keep the warranty card, proof of purchase of the product and the label (label) of the product that is included in the packaging of each product. The buyer is obliged to properly check and inspect the goods upon receipt. In the event that it finds defects in the product, it is obliged to notify the seller of this fact without undue delay. The manufacturer is not responsible for defects caused by the transport of the goods by the buyer's means of transport, as well as for later complaints about the number of delivered goods or damage to the shipment that was not stated on the receipt of the goods. Contact details for filing a complaint are listed on the sales document. In the event of a complaint, please proceed in accordance with the seller's Complaints Policy, which can be found on the seller's website or directly from the seller.

Statutory warranty: Provided that the principles of proper use and care of the product are observed in accordance with these Instructions for Use, the right to defect (complaint) must be exercised by the buyer within 24 months (unless otherwise stated) from the receipt of the product when purchased from the seller, and if the product is not personally collected from the seller, the receipt of the product is understood as the moment when the buyer took over the product from the carrier. The length of the right arising from defective performance (hereinafter referred to as the warranty period or warranty) is extended by the period for which the product was under warranty repair, until the time when the buyer was obliged to take over the product after the end of the complaint procedure. The warranty period is not to be confused with the lifetime of the product, i.e. the period for which, with proper use and proper care, including maintenance, the product can last due to its properties, given purpose and differences in the intensity of its use. The warranty period is a general administrative period that has no direct connection with the normal lifespan of a particular product. Upon expiration of the warranty period, the purchaser's rights to claim the warranty cease to exist. The warranty does not cover normal wear and tear of the positioning aid caused by its normal use. Furthermore, liability for defects in products does not apply to defects that the buyer was notified of by the seller at the time of purchase (conclusion of the contract) and at the same time did not refuse to accept the product when delivering it. Furthermore, the right from defective performance cannot be applied to products sold at lower prices, for which a lower price was agreed precisely with regard to their defects, which the buyer was aware of before the purchase of the products.

Extended Warranty: The manufacturer provides purchasers with an extended warranty for core defects on selected positioning aids under the conditions set forth below, with the duration of the warranty for defects being indicated on the label or sticker on the packaging of the device. An extended warranty for core defects is provided for selected aids, conditional on placing the device on a flexible base (slatted grate). There is no charge for this benefit, you get the extended warranty for free, you only need to register your aid. It is important that the buyer registers no later than 30 calendar days from the date of purchase on the <https://reg.hilding.cz>. To register, you only need to copy the name, size and serial number (SN code) – you can find it on the paper self-adhesive label located on the packaging. Further details about the extended warranty can be obtained from the dealer. The extended warranty for defects applies only to the core, where when replacing the core, a warranty is provided for the new one for the duration of the remaining extended warranty, but at least 24 months.

By purchasing a product – positioning aid, the buyer confirms that he has been properly acquainted with the seller's complaint procedure. The claimed device must be complete and in complete condition, it must be in a hygienically acceptable condition (i.e. in any way unsoiled, dry, hygienically harmless, etc.), preferably in the original or appropriate replacement packaging, suitable for the transport of goods (caution – stretch foil in a thin layer is not considered suitable packaging for transport!). In the event that the complained device is hygienically unsafe (dirty, wet, smelly, etc.), the complaint will be suspended until the buyer removes the hygienic defect and this fact will be marked in the complaint protocol. However, if the buyer fails to deliver a hygienically safe aid to the complaint, even within 1 month from the date of interruption of the complaint, the complaint is considered to be terminated due to its groundlessness.

Report any serious adverse event that has occurred in connection with the device concerned to the manufacturer and the competent authority.

Last revision of this document: 2025-05-22, rev. 3, language version EN

Hilding Anders Česká republika a.s. has prepared questionnaires for the purpose of collecting clinical data within the framework of PMCF (Post-Market Clinical Follow-up). The aim of data collection is to confirm the safety and effectiveness of anti-decubitus mattresses and monitor potential risks and adverse effects. This assessment is an important criterion that must be met in accordance with the legislative requirements for medical devices. The data obtained from the questionnaires will be used to update the clinical evaluation in accordance with Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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